Manual Kolibri S542

Pegasus-Mobility A/S

Nørregade 7
5450 Otterup
Telefon 70 27 27 18

Serienummer: Salgsdato/garantidato:
--

INDHOLDSFORTEGNELSE

Indhold

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SGS		No: HE1V018/2016
		Page: 1 of 1
VERIFICATION OF COMPLIANCE		
Verification Report No.	HE1V018/2016	
Date of Issued	March 18, 2016	
Applicant	MERITS HEALTH PRODUCTS CO., LTD	
Address of Applicant	NO.18,JINGKE ROAD, T.P.M.T PARK, NANTUN DISTRICT, TAICHUNG CITY, TAIWAN, R.O.C.	
Manufacturer	MOBILE ENERGY TECHNOLOGY CO., LTD	
Address of Manufacturer	NO.13, KONG 9TH ROAD, 2ND INDUSTRIAL PARK, LINKOU, TAIPEI, R.O.C.	
Product	7S4P LI-Ion Battery Pack	
Model/ Type Ref.	MER-01	
Ratings and Principal Characteristics	24 Vdc, 274.4 Wh UN ST/SG/AC.10/11/Rev.5/Amend.1 Manual of Tests and Criteria, Part III, sub-section 38.3	
Tested According to	IEC 62281 Edition 2.0:2012 Safety of primary and secondary lithium cells and batteries during transport	
As shown in the Test Report No. which forms Part of this Verification	HE10018/2016	
Conclusion		
Based upon a review of the Test Report(s), the tested sample of the product mentioned above is deemed to comply with the below requirements of:		
UN ST/SG/AC.10/11/Rev.5/Amend.1 Manual of Tests and Criteria, Part III, sub-section 38.3 IEC 62281 Edition 2.0:2012: Safety of primary and secondary lithium cells and batteries during transport		
Test 1: Altitude Simulation	Test 4: Shock	
Test 2: Thermal Test	Test 5: External Short Circuit	
Test 3: Vibration	Test 7: Overcharge (Pack Only)	
Note : This verification is only valid for the product and configuration described and in conjunction with the technical data detailed above.		
Authorized Signatory:		
		
SGS Taiwan LTD. Terence Hsieh Department Manager		
<small>Unless otherwise stated the results shown in this test report refer only to the sample(s) tested and such sample(s) are retained for 90 days only. This test report cannot be reproduced, except in full, without prior written permission of the Company. 除非另有說明，此報告僅針對測試之樣品負責，且測試樣品僅保留 90 天。本報告未經本公司書面許可，不可隨意複製。 This document is issued by the Company subject to its General Conditions of Service printed overleaf, available on request or accessible at www.sgs.com/terms_and_conditions.htm and, for electronic format documents, subject to Terms and Conditions for Electronic Documents at www.sgs.com/terms_e-documents.htm. Attention is drawn to the limitation of liability, indemnification and jurisdiction issues defined therein. Any holder of this document is advised that information contained hereon reflects the Company's findings at the time of its intervention only and within the limits of Client's instructions, if any. The Company's sole responsibility is to its Client and this document does not exonerate parties to a transaction from exercising all their rights and obligations under the transaction documents. This document cannot be reproduced except in full, without prior written approval of the Company. Any unauthorized alteration, forgery or falsification of the content or appearance of this document is unlawful and offenders may be prosecuted to the fullest extent of the law. No.31, No.33, No.35, Wu Chyuan Road, New Taipei Industrial Park, Wuku District, New Taipei City, Taiwan/新北市民權區新北產業園區五權路 31 號, 33 號, 35 號 No.134, Wu Kung Road, New Taipei Industrial Park, Wuku District, New Taipei City, Taiwan/新北市民權區新北產業園區五工路 134 號 SGS Taiwan Ltd. No.134, Wu Kung Road, New Taipei Industrial Park, Wuku District, New Taipei City, Taiwan/新北市民權區新北產業園區五工路 134 號 台灣檢驗科技股份有限公司 t (886-2) 2299-3279 f (886-2) 2299-9558 www.taipei.sgs.com Member of SGS Group</small>		

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EU Declaration of Conformity

Manufacturer:

MERITS HEALTH PRODUCTS CO., LTD.
NO. 18, JINGKE RD., NANTUN DISTRICT,
40852 TAICHUNG CITY, TAIWAN

whose single Authorized Representative:

MedNet EC-REP GmbH
Borkstrasse 10, 48163 Münster, Germany

We, the manufacturer, herewith declare under our sole responsibility that the products

Electrically Powered Scooters

**Model No.: S940A, S930A, S941A, S836E, S846E, S846G, S846H, S846I, S846K,
S837, S847, S337, S3379, S344, S148, S745, S745A, S745B, S840E, S840T,
S542A, S247A S/N : XXXX**

(including system components and accessories)

**UMDNS-Code: 45684; GMDN-Code/Preferred Terms: 45684 / Assistive scooter,
electric-motor-driven**

Intended Purpose: A battery-powered, electric-motor-driven mobility device designed to be operated by a person with a disability and consisting of a seat over two rear wheels, a platform for the feet, and handlebars in front to turn one or two steerable front wheels connected via a steering-column. The device is used to enhance the user's ability for self-transportation in the local environment and/or to promote the user's physical/recreational activities.

The classification is made in accordance with **MDR (EU) 2017/745, Annex VIII Section 4, 4.1, rule 1, "All non-invasive devices are classified as class I, unless one of the rules set out hereinafter applies" & Section 6, 6.5, rule 13, "All other active devices are classified as class I."** It bears the mark

To which this declaration relates is in conformity with the standard(s) or common specification or other normative document(s) specified in the Annex on the following pages.

Compliance of the designated product with the MDR (EU) 2017/745 has been assessed.

following the provision of Regulation: EU Medical Device Regulation [MDR (EU) 2017/745] ANNEX II (Technical documentation), ANNEX III (Technical documentation on post-market surveillance), and including an assessment of the technical documentation; RoHS 2 of Directive 2011/65/EC

This Declaration of conformity is valid in connection with the release document for the respective batch of produced devices.

The above mentioned declaration of conformity is exclusively under the responsibility of

MERITS HEALTH PRODUCTS CO., LTD.

Taichung, Taiwan

2021/03/08

Place, date

Martin Tseng

Supervisor of R&D Dept.

Legally binding signature, Function

Martin Tseng
2021/03/08

EU Declaration of Conformity
DOC-S01-04

Page 1/5